


Step by step


Before starting the mattress, the patient should be **positioned in supine** (face up, centered in bed).

Use positioning cushions, if needed. Ensure that heels are properly offloaded.

 **To start the turning program, press start.**

The mattress will begin curving. This process will take approx. 35 mins, and then the continuous turning starts.

For patient care mode (static), press the start button once more while the program is running. This activates a 30 min. stop.

 **To stop the turning program, press stop.**



Standard settings gives 20° turning degree both ways, at medium turning speed and a 15 min delay on each side.



See our instruction video

QUICK GUIDE HOW TO OPERATE

Tidewave™ Turning Mattress

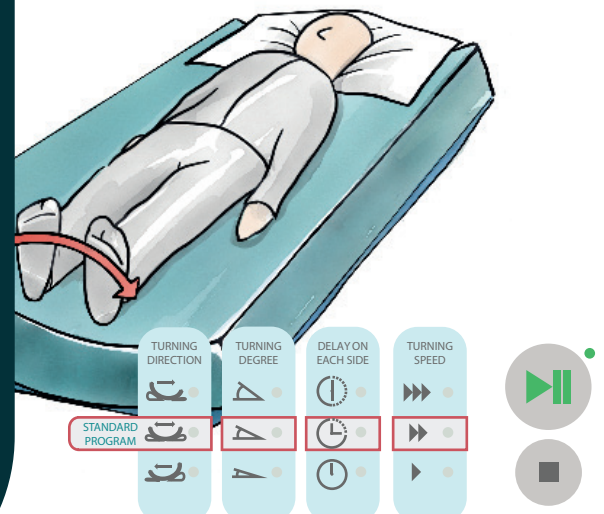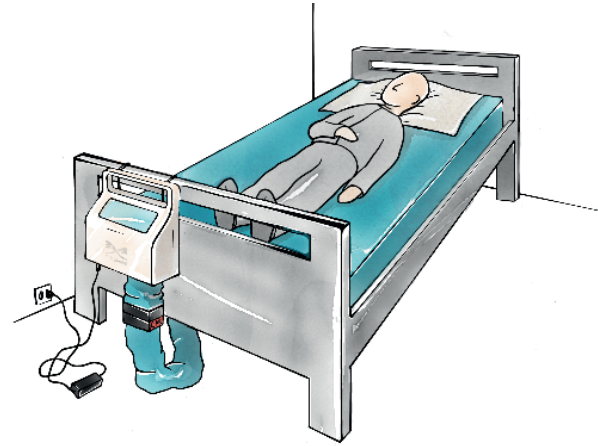




Individualization

Turning Direction

It is possible to exclude the patient's sides from being loaded.




TURNING DIRECTION

-  Exclude the patient's right side
-  Turn in both directions
-  Exclude the patient's left side

Turning degree

Adapt the turning degree to the patient's health condition.




TURNING DEGREE

-  Patient is turned to 30°
-  Patient is turned to 20°
-  Patient is turned to 10°

Delay on each side

The turning can be delayed at the end of a turn based on need.




DELAY ON EACH SIDE

-  30 minutes delay on each side
-  15 minutes delay on each side
-  Continuous movement, no delay

Turning speed

Choose between 20–60 minutes from one side to the other.

TURNING SPEED

-  High speed: 20 minutes
-  Medium speed: 40 minutes
-  Low speed: 60 minutes




Custom settings

To choose a setting, press the round button accompanying the individual setting, one to three times. The active setting lights up on the panel.



See our instruction video

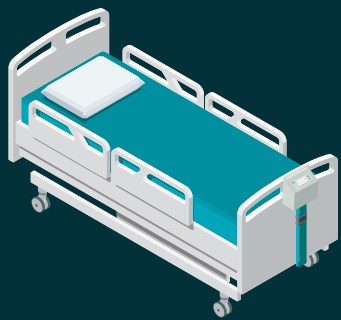

Step by step

Place the mattress in bed with the hose towards the foot end. With the foam side pointing up.

Guide the hose through the base of the bed. Make sure it is not squeezed or pinned.

Attach the pump unit to the foot end of the bed and connect it to the mattress (audible click).

Connect the power cord to the pump unit and then to a wall socket. Then wait 30 seconds.




After 30 seconds the panel on the pump unit will light up. The mattress is now ready for use.




See our
instruction video

QUICK GUIDE INSTALLATION

Tidewave™ Turning Mattress

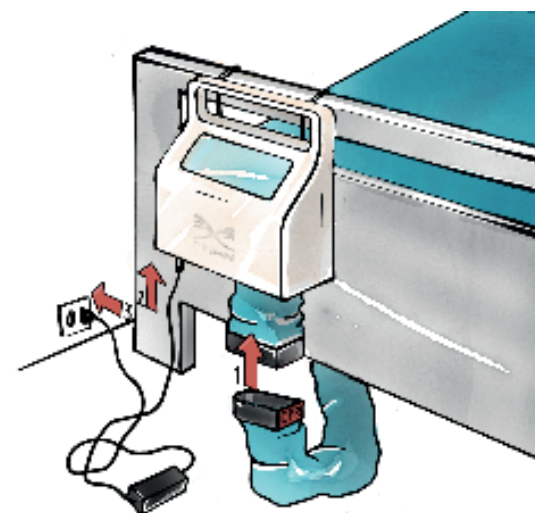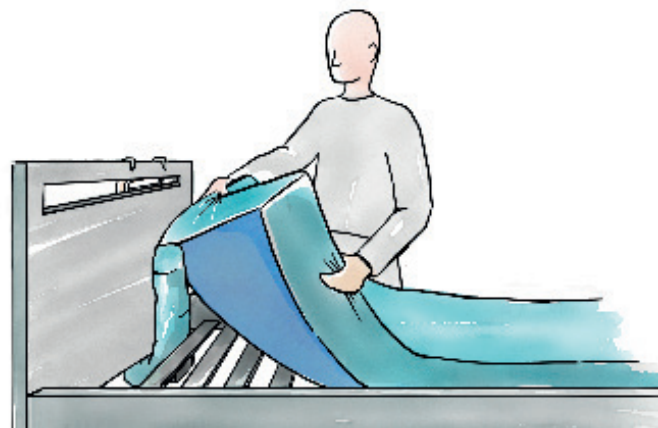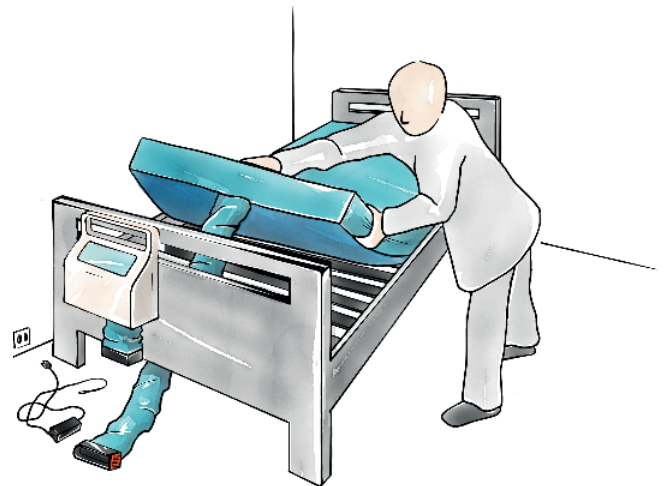

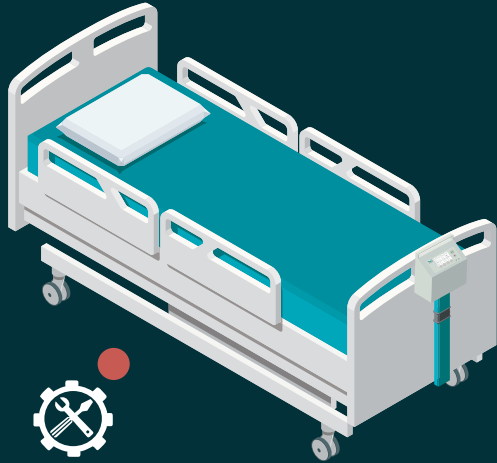

Check connector

Check if the air hose is connected, pull it gently to make sure the connector is properly engaged.

Check cable

Make sure the air hose is not in a tight bend or pinched between the mattress and bed frame.



More than 30° backrest

If the backrest is raised above 30°, the turning program will stop and the >30° lights up.




When the backrest is lowered to 30° or below, the mattress will automatically resume.

Check the user manual

If none of the above seem to be the problem, check out the troubleshooting section of the user manual or call your service technician.

QUICK GUIDE TROUBLESHOOTING

Tidewave™ Turning Mattress

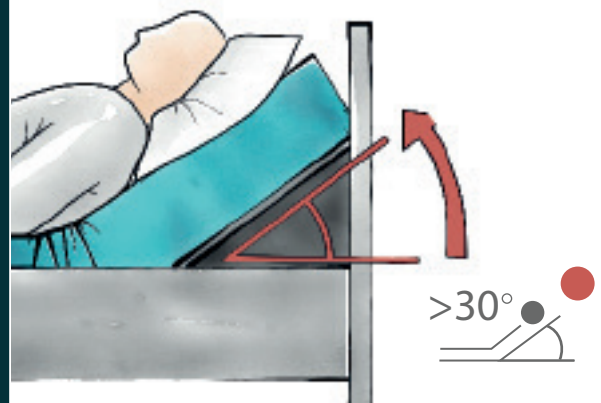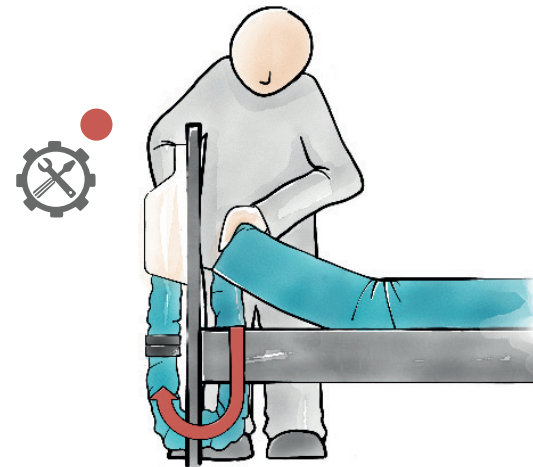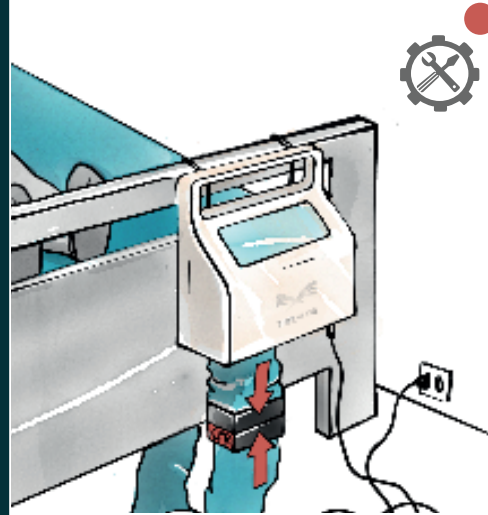

See our user manual